

Sysmex Corporation

1-5-1, Wakinhama-Kaigandori,
Chuo-ku, Kobe, Hyogo 651-0073, Japan
Tel +81 (78) 265-0500
Fax +81 (78) 265-0524
www.sysmex.co.jp

Sysmex Asia Pacific Pte Ltd

9, Tampines Grande #06-18,
Singapore 528735, Singapore
Tel +65 6221-3629
Fax +65 6221-3687
www.sysmex-ap.com

Sysmex India Pvt. Ltd

1002, Damji Shamji Business Galleria, 10th Floor, LBS
Marg Kanjurmarg (West), Mumbai 400 078, India
Tel +91 (22) 6112-6666
Fax +91 (22) 2577-6790
www.sysmex.co.in

PT Sysmex Indonesia

Cyber 2 Tower, 5th Floor, Unit E, Jl. HR. Rasuna Said,
Blok X5 No. 13, Jakarta Selatan 12950, Indonesia
Tel +62 (21) 3002-6688
Fax +62 (21) 3002-6699
www.sysmex.co.id

Sysmex (Malaysia) Sdn Bhd

11A & 15, Jalan PJS 7/21, Bdr Sunway,
47500 Subang Jaya, Selangor, Malaysia
Tel +60 (3) 5637-1788
Fax +60 (3) 5637-1688
www.sysmex.com.my

Sysmex (Thailand) Co., Ltd

12th Floor, Tonson Tower, 900 Ploenchit Road,
Lumpini, Pathumwan, Bangkok 10330, Thailand
Tel +66 (2) 635-5777
Fax +66 (2) 257-0530
www.sysmex.co.th

Sysmex Philippines Inc.

30th Floor MDC 100 Bldg, E. Rodrigues Jr. Ave, cor
Eastwood Ave, Bagumbayan, Quezon City, Philippines
Tel +63 (2) 621-2460
Fax +63 (2) 621-2432
www.sysmex.com.ph

Sysmex New Zealand Ltd

382 - 386 Manukau Rd, PO Box 26-085, Epsom,
Auckland 1344, New Zealand
Tel +64 (9) 630-3554
Fax +64 (9) 630-8135
www.sysmex.co.nz

Sysmex Vietnam Co., Ltd

Unit 802, Centre Point Bldg, 106 Nguyen Van Troi St,
Phu Nhuan Dist, Ho Chi Minh City, Vietnam
Tel +84 (8) 3997-9400
Fax +84 (8) 3997-9405
www.sysmex.com.vn

Sysmex Vietnam Co., Ltd (Hanoi Branch)

Floor 10, Vinaconex Building, 34 Lang Ha St,
Dong Da Dist, Ha Noi, Vietnam
Tel +84 (4) 3776-7020
Fax +84 (4) 3776-7022
www.sysmex.com.vn

Sysmex Australia Pty Ltd

Level 9, Avaya House, 123 Epping Road
North Ryde NSW 2113, Australia
Tel +61 (2) 8875-7700
Fax +61 (2) 8875-7777
www.sysmex.com.au

About Sysmex

Sysmex is a global market leader in the development and implementation of clinical diagnostic and health IT products and services for laboratories, hospitals and healthcare organizations.

We deliver total solutions in the field of clinical laboratory testing, including hematology, hemostasis, urinalysis, clinical chemistry, immunoassay system and life science. Supplying products and services to customers in more than 150 countries, Sysmex is the market leader in hematology and coagulation instrumentation worldwide.

By working together with our customers and business partners, Sysmex is committed to our mission of shaping the advancement of healthcare.



Fully Automated Coagulation Analyzer

CS-2400 / CS-2500

The Analyzer you know becomes better

**Successor of the CS-series,
based on proven track record of
CS-2000i/2100i & CS-5100 and even more...**


A high performance coagulation system that allows laboratories to perform routine and specialized testing on a single analyzer.



INNOVATIVE TECHNOLOGY

Delivering high quality results on every test run

With the release of the IVD industry's first coagulation analyzer having a multi-wavelength analysis technology, Sysmex aims to help improve operational efficiency and the quality of coagulation testing.

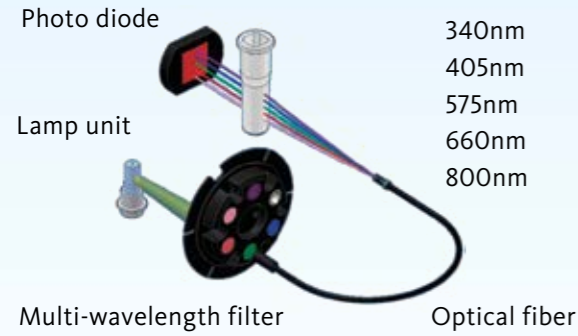


Multi-wavelength detection system

The CS-2400/CS-2500 is equipped with an optical fiber that supplies light at 5 different wavelengths, and a detector capable of receiving light in multiple wavelengths.



Having this multi-wavelength capability, the system is not only able to care of your analysis results, but also your pre-analytical check function.

The power of multi-wavelength technology

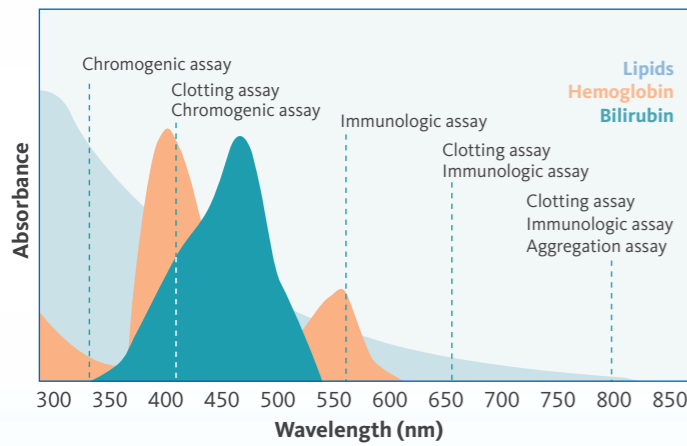

Spectra of interfering substances

- Measure multiple parameters with varying methodologies simultaneously. Throughput is improved as the time to measure a single sample is reduced.
- Reduce the effect of interfering substances on the absorbance spectra by utilizing optimal wavelengths. Optimal wavelength is automatically selected based on sample characteristics (wavelength switching) for more accurate results.
- The system automatically switches to the secondary wavelength when the coagulation reaction shows only a small change at the primary wavelength. Such as in the case of a severely lipemic or low fibrinogen sample.

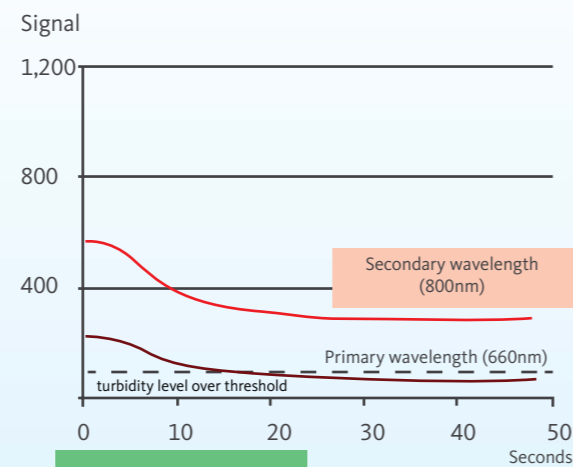
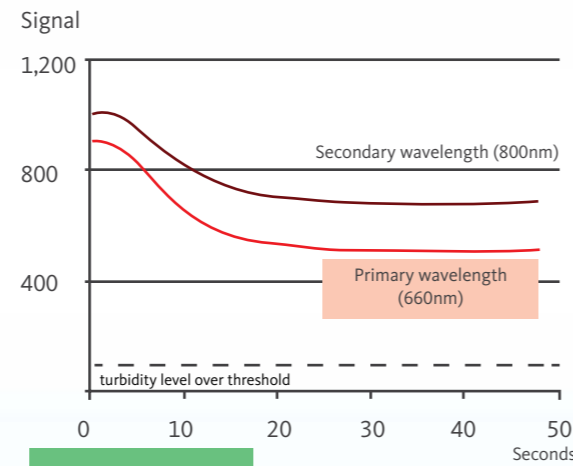


Wavelength switching during PT measurement

Hemolysis and inappropriate filling of tubes are 2 of the most common pre-analytical problems in coagulation testing¹. The CS-2400/CS-2500 are the first coagulation analyzer to incorporate the pre-analytical check function to minimize pre-analytical errors by automatically detecting and managing unsuitable samples prior to any testing.


HIL check for interfering substances


- Utilizing the multi-wavelength technology, the system measures the level of Hemolysis (H), Icterus (I), and Lipemia (L) in the sample with a HIL detector at 3 wavelengths - 405, 575 and 660 nm.
- The level of HIL can be displayed as a flag, based on user-defined criteria for each parameter, to ensure appropriate interpretation of results.

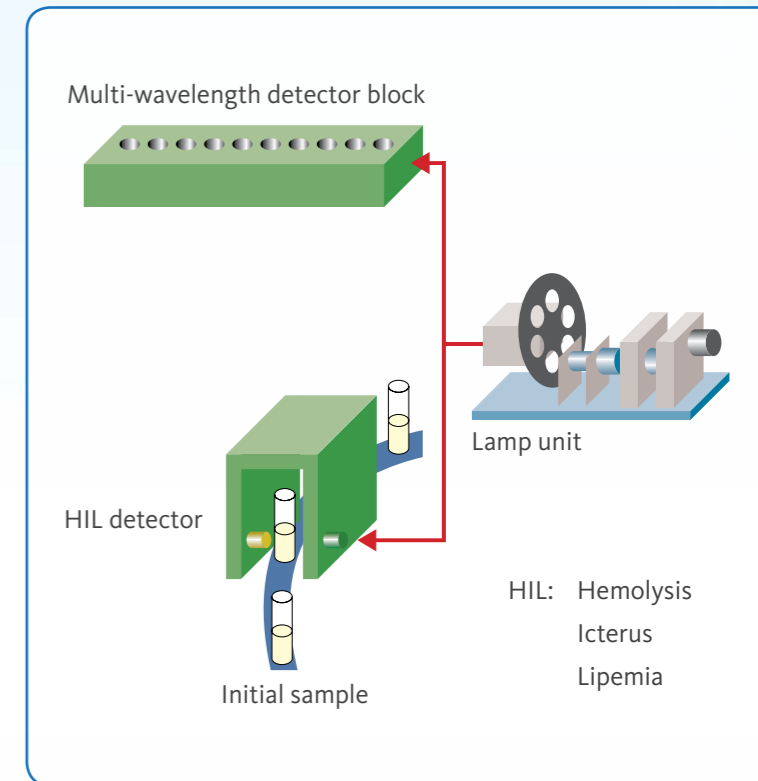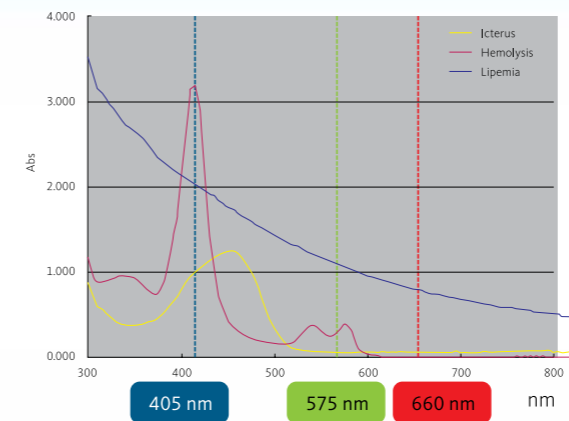

Primary sample volume check

- When aspirating a sample for the first time, the system detects the level of liquid in the sample tube. It then determines whether or not it contains an appropriate volume of blood according to the CLSI guideline of 9:1 ratio.
- Sample tubes with volumes outside the acceptable range are flagged to avoid reporting of erroneous results.

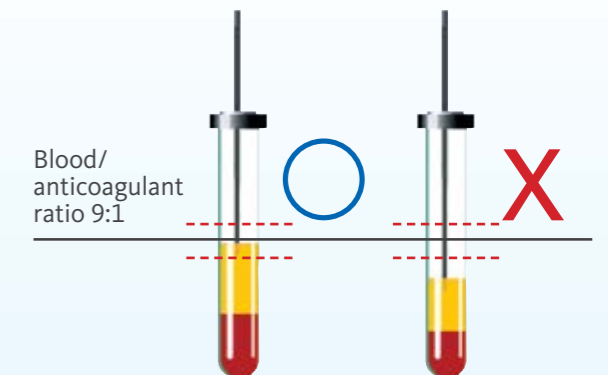

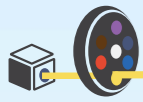


USABILITY

Designed with you in mind

We are focused to enhance your experience with the Sysmex systems you use day in, day out. The CS-2400/CS-2500 combines proven and successful characteristics of its predecessors with new features to further increase the ease of use and improve efficiency within your laboratory.

Intuitive cap-piercing technology

- Sysmex has accumulated more than 20 years of experience in developing cap-piercing systems.
- The system can handle various types of blood collection tubes. It automatically detects the presence of a cap enabling measurement, even when capped and uncapped collection tubes are placed on the same rack.

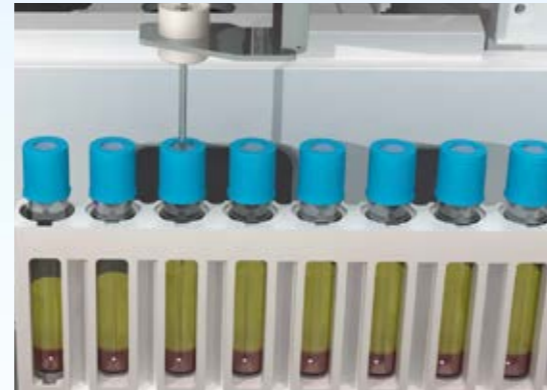

Cap-piercing on CS-2500
*CS-2400 is a non cap-piercing model

Enhanced reagent management functions

- Can load up to 40 reagents.
- An in-built barcode reader automatically identifies and registers the reagent, no matter where the reagent is placed.
- Advance software that:
 - displays reagent information (name, lot number, expiry date, etc.) on the reagent screen;
 - offers real-time tracking of remaining volume, or number of tests;
 - displays reagent on-board time.



Reagent screen

Convenient reagent loading

Reagents can be replaced or added without interrupting the current measurement.



Reagent tables

Extended on-board reagent stability

- Reagent racks are designed so that reagent vials are tilted to reduce dead volume and maximize the number of tests per vial.
- The reagent table is cooled at 10 °C to allow longer on-board stability and support 24-hr laboratory operations.



Tilted reagent rack

Comprehensive QC program


- Automatically manage QC measurement by placing QC material in a rack that is registered as a QC rack.
- Automatic QC performed at regular intervals.
- QC vial enables QC checking before a new vial of reagent is used for testing.
- Following the setting guide allows for easy registration of a new lot of QC.



QC registration

Complete measurement information


- The job list screen display the:
 - sample information;
 - estimated measurement completion time;
 - measurement status;
 - measurement results for each order, allowing measure information to be checked at a glance.
- Measurement results, reagent lot number, calibration curve used in testing, and the date/ time of the recent QC measurement can all be traced easily.

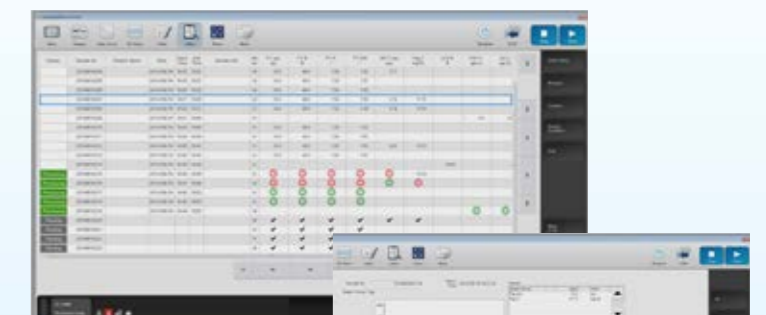

Measurement results and status



Sample information

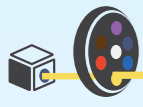


FUNCTIONS WITH HIGH ADDED VALUE

Meeting all your hemostasis needs in one platform

We recognized the needs of today's progress laboratories. As such the CS-2400/CS-2500 is designed to perform a broad range of routine and specialized assays. It supports specialized tests such as labor intensive platelet aggregation assays as well as improve efficiency of routine tests such as cross mixing tests.

Automated platelet aggregation Assay

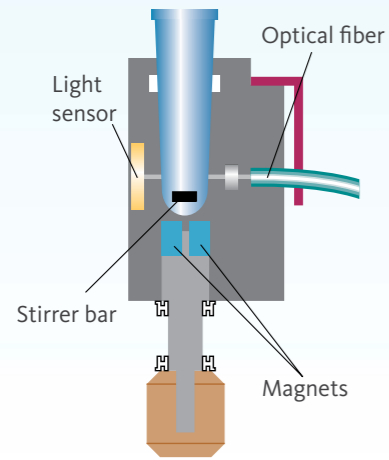

Supports aggregation method in 4 channels




Sample tubes containing stirrer bars

Provide assessment of platelet-based ristocetin cofactor activity (vWF:RCo) and platelet aggregation* to ADP, epinephrine, collagen, arachidonic acid and ristocetin².

* Available in 2nd half of 2015

Cross-mixing testing

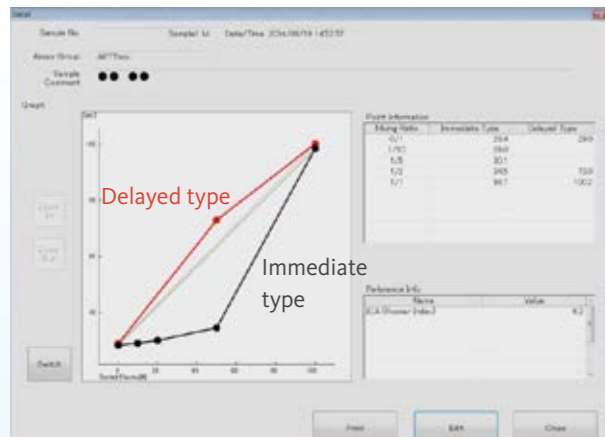


ICA (Rosner Index) = Index of circulating anticoagulant
 $ICA = (b-c) / a \times 100$
a = Coagulation time of 100% patient serum
b = Coagulation time of 50% patient serum
c = Coagulation time of normal serum

Browser details screen (cross mixing test results)

Simplify cumbersome procedures

- Automatic dilution
- Graph creation
- Index calculation
- Overlay of immediate and delayed type of graphs

Clot waveform analysis (for research)

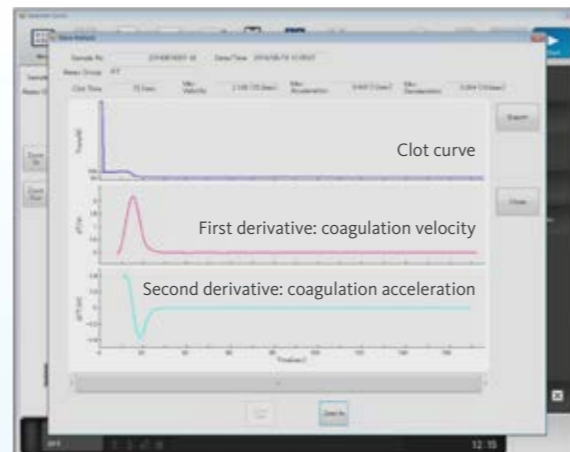

Wave analysis screen

Potential clinical utility for evaluating coagulation function in various bleeding disorders, especially Hemophilia A, B, and monitoring bypass therapy³.

References

1. Lippi G, Salvagno GL, Montagnana M, Lima-Oliveira G, Guidi GC, Favaloro EJ. Quality standards for sample collection in coagulation testing. *Semin Thromb Hemost.* 2012; 38:565-75.
2. Lawrie AS, Kobayashi K, Lane PJ, Mackie IJ, Machin SJ. The automation of routine light transmission platelet aggregation. *Int J Lab Hematol.* 2014; 36(4):431-8.
3. Shima M, Thachil J, Nair SC, Srivastava A. Towards standardization of clot waveform analysis and recommendations for its clinical applications. *J Thromb Hemost.* 2013; 11:1417-20.